


**THE AVAILABLE CYCLES** (complete list in the user manual)

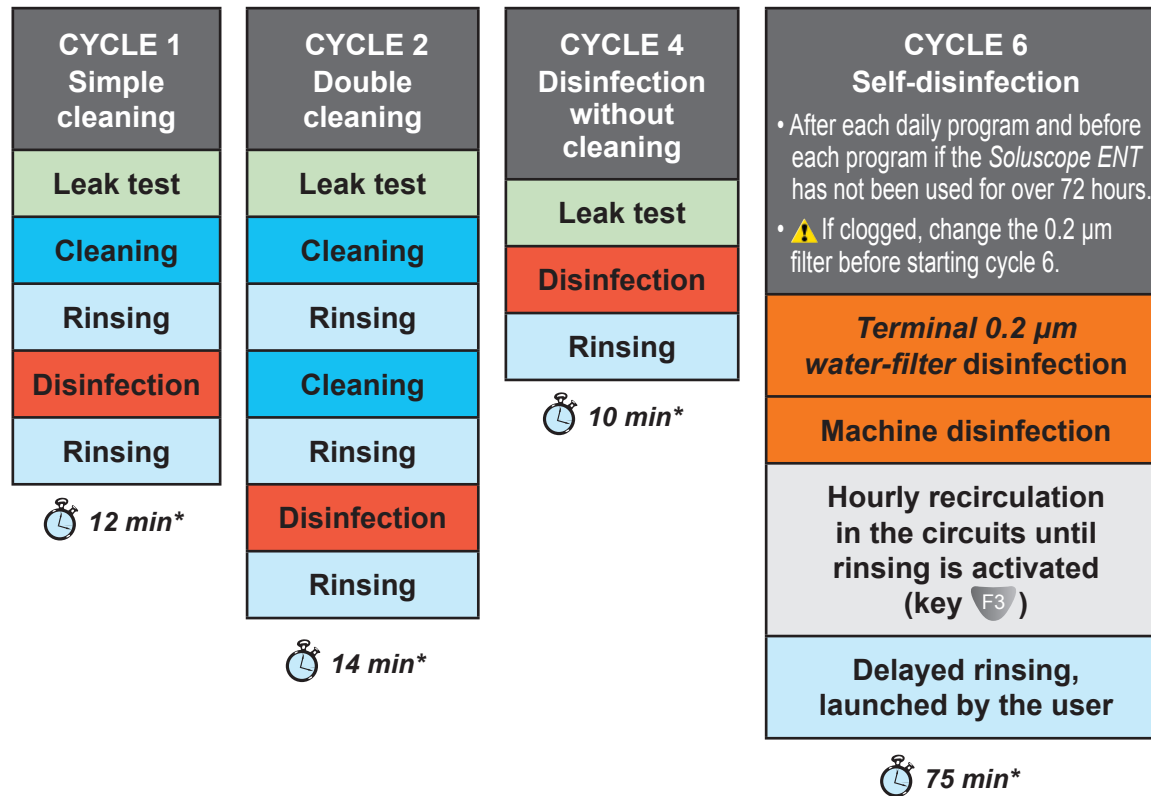

The instructions on this page do not exempt the user from reading the user manual.


\* Cycle times may vary according to water network quality, endoscope type, filter clogging and water temperature.

**LAUNCHING A CYCLE**

As soon as the endoscopy is finished, the endoscope must be pretreated before being loaded into the *Soluscope*. A minimum pretreatment protocole is presented in the user manual. Connect the endoscope to the *Soluscope ENT* with the proper connectic.

For cycle 6 and 8 : connect

From main menu (press **MAIN** to display it).

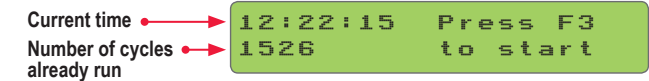

Press **F3** and carefully read intructions on display.

Once the input is complete, the following message is displayed :



Endoscope identification

Close the lid without forcing and press **F3** to start.

**MAIN** to re-enter the data

**F1** to interrupt the cycle