

THE AVAILABLE CYCLES (complete list in the user manual)

REPROCESSING CYCLES

CYCLE 1 Simple cleaning	CYCLE 2 Double cleaning + alcohol flush **	CYCLE 4 Disinfection without cleaning
Leak test	Leak test	Leak test
Pre-rinse	Pre-rinse	Pre-rinse
Cleaning	Cleaning	Disinfection
Rinsing	Rinsing	Rinsing
Disinfection	Cleaning	Purging
Rinsing	Rinsing	15 min *
Purging	Disinfection	
20 min *	Rinsing	
	Purging	
	Alcohol injection**	
	Purging**	
	22 min *	

OTHER CYCLES

CYCLE 6 Self-disinfection	CYCLE 8 Descaling
<ul style="list-style-type: none"> At least once every 24 hours or after each daily program. ⚠ If clogged, change the filters before starting cycle 6. 	Descaling
Chlorine disinfection: Disconnection Tank + Pre-filters unit + 0.2 µm terminal water filter + Machine	Rinsing
Rinsing	Disinfection
Glutaraldehyde disinfection: Machine + 0.2 µm terminal filter	Rinsing
Hourly recirculation in the circuits until rinsing is activated (key F3)	Purging
Delayed rinsing, launched by the user	30 min *
Purging	
50 min *	CYCLE 5 Additional purging
	Purging
	Alcohol injection**
	Purging**
	4 min *



Purging alone allows the reuse of the endoscope quickly. For longer storage (overnight or longer) the endoscope channels should be flushed with 0,2 µm filtered air.




The instructions on this page do not exempt the user from reading the user manual.

* The cycle time may vary according to the type of endoscope, the filters clogging (+/- 2 min) and the temperature of the inlet water.
 ** For Soluscope 3 with the alcohol option and if activated.

LAUNCHING A CYCLE

As soon as the endoscopy is finished, the endoscope must always be pretreated before being loaded into the Soluscope. A minimum pretreatment protocol is presented in the user manual.

Connect the endoscope to your Soluscope using the connectic appropriate (matching the endoscope number).

For cycle 6 and 8 : connect 

From main menu (press **MAIN** or **F1** to display it).

Current time → 12:22:15 Press F3
 Number of cycles already run → 1526 to start

Press **F3** and carefully read instructions on display.

Once the input is complete, the following message is displayed :

Chosen cycle → Cycle 1 / F3 Start
 1257 OL GAST 201692

Endoscope identification →

Close the lid without forcing and press **F3** to start.

FOR CYCLE 6:

First, add 6 chlorine tablets into the bowl, and close the lid.
Wait for the following display and insert 6 chlorine tablets into the break tank.



Chlorine inserted?
 F1: No F3: Yes

MAIN to re-input the data

F1 to interrupt the cycle