



Reminder: All maintenance must be carried out by a *Soluscope* certified technical personnel.

DAILY MAINTENANCE

- D1** Check the opening dates marked on the cans *Soluscope A*, *Soluscope P* and *Soluscope C+*. These chemicals must be used within two months after the opening date.
- D2** Clean the flat surface surrounding the bowl, the inner sides of the bowl, the lid (inside and out) as well as the lid seal with a disinfecting surface spray. **Use neither acetone nor any other chemical.**
- D3** Make sure that the last printouts do not indicate *FILTERS ALMOST CLOGGED*. In this case, change the filters before starting cycle 6.
- D4** Launch the self-disinfection cycle 6 once you have carefully read the instructions in the relevant section, chapter 5 of your *Soluscope ENT* user manual.
- D5** Dry and check the connectics and disinfect their connectors with a surface disinfectant spray. Make sure that they are not kinked. Lubricate the gasket on the leak test connector with silicone oil. Store the connectics in a dry and clean place.
- D6** Ensure that all tickets printed out by the machine during the day have been collected and filed, **including tickets from unsuccessful cycles.**



Defective connectics must be immediately condemned and withdrawn from use.

WEEKLY MAINTENANCE *(to be carried out before the daily maintenance)*

- W1** Check that the 0.2 µm terminal water filter has not reached the use-by date, if so, replace all the filters.
- W2** Check that the air-filter has not reached the use-by date. If so, replace the filter.
- W3** Check that the appendix ***Maintenance recording form*** has been correctly filled in by the user.

MONTHLY MAINTENANCE

- M1** Take a rinsing water sample from your *Soluscope ENT* in accordance with the appendix ***Instructions to sample rinsing water***. Contact your distributor.

YEARLY MAINTENANCE OR 4000 CYCLES *(whichever comes first)*

- Y1** For a preventive maintenance of your *Soluscope ENT*, contact your distributor.
- Y2** We recommend having your *Soluscope ENT* requalified in accordance with the *ISO EN 15883-4* standard conditions concerning washers-disinfectors. Contact your distributor.



Should you notice white deposits on the endoscopes or in the bowl, check the total hardness (TH degree) of the water and contact your distributor.

PROLONGED STOP

- PS1** If your *Soluscope ENT* has to be stopped for 72 hours or more, contact your distributor for instructions on how to proceed to this prolonged stop.