



Reminder: All maintenance must be carried out by a *Soluscope* certified technical operators. Failure to follow these instructions can avoid the manufacturer's warranty and compromise the disinfection cycle effectiveness.

DAILY MAINTENANCE

- D1** Check the opening dates marked on the *Soluscope D* and *Soluscope C+* product cans. They must be used for a maximum of 2 months after opening.
- D2** Brush the grid at the bottom of the bowl in a recommended cleaning solution, rinse and replace.
- D3** Clean the lid, the top of the bowl and the inside over 5 cm with a surface disinfecting spray. **Do not use acetone or any other solvent.**
- D4** Clean the lid seal with a surface disinfecting spray. See **Lid seal inflation - Cycle 10**, chapter 5 in the *Soluscope 3* user manual.
- D5** Launch a cycle 6 for self-disinfection once having checked the need for filter replacement. See **Soluscope 3 self-disinfection - Cycle 6**, chapter 5 in the user manual.
- D6** Dry and check the connectics and disinfect their connects with a surface disinfectant spray. Check that there are no holes or kinks. Lubricate the gasket on the leak test connect with silicone oil. Store the connectics in a dry and clean place.
- D7** Ensure that the tickets printed by the machine during the day have been collected and filed, **including tickets from unsuccessful cycles**. Keep them away from light.



Defective connectics must be immediately removed from use and replaced.

WEEKLY MAINTENANCE (to be carried out before the daily maintenance)

- W1** Check that the use life date on the 0.2 µm water filter has not expired. If so, replace it and the pre-filter unit cartridges at the same time.
- W2** Check that the use life date on the *HEPA* air filter has not expired. If so, replace it.
- W3** Launch cycle 8 for descaling (see **Descaling of the Soluscope 3 - Cycle 8**, chapter 5 in the *Soluscope 3* user manual).
- W4** Ensure that the appendix **Maintenance recording form** has been correctly filled in by the users.
- W5** Carry out the daily maintenance.

MONTHLY MAINTENANCE

- M1** Take a sample of the last rinse water from the *Soluscope 3* in accordance with the appendix **Rinse water sampling instructions**.

YEARLY MAINTENANCE OR 2000 CYCLES (whichever comes first)

- Y1** Have your *Soluscope 3* serviced. Contact your authorised distributor.
- Y2** Have your *Soluscope 3* re-qualified in accordance with its dedicated standards (*ISO/DIS15883-4*). Contact your authorised distributor.



Should you detect white deposits on the endoscope or in the bowl, check the water hardness (TH degree) and contact your distributor.

PROLONGED STOP

- PS** If your *Soluscope 3* has to be stopped for 72 hours or more, contact your authorised distributor for instructions on how to proceed to this prolonged stop.