

DAILY :

- D1. Check the opening dates that are marked on the Soluscope-P, Soluscope-A, and Soluscope-C bottles. The products are not to be used more than 2 months after opening.
- D2. Brush the grid of the tank in a decontaminating cleaning product, and re-install it properly.
- D3. Clean all around the tank (flat surface), and the inside on about 5 cm with alcohol. **Do not use acetone.**
- D4. Check the possible need for 0.2µm water filter changing (See "Filter almost blocked" on the last endoscope reprocessing ticket), anyway launch a cycle of self-disinfection 6
- D5. Dry the connectics and disinfect their connectors with a surface disinfectant spray. Check for the presence of any hole, crack or slit. Lubricate the leak test connectics with silicone oil. Store connectics in a dry and clean place. **Any defective connectic must be replaced immediately.**
- D6. Make sure that the tickets printed by the machine during the day have been taken out and filed. **Including unsuccessful cycles tickets.**

WEEKLY , PRIOR TO THE DAILY MAINTENANCE :

- W1. Check the date on the 0.2µm water filter. It must not be over 90 days old.
- W2. Check the date on the prefiltering unit cartridges. It must not be over 90 days old **(Replace the two of them at the same time).**
- W3. Check the date on the air filter. **It must not be over 1 months old.**
- W4. Check that the maintenance record sheet is correctly filled in by the users.

MONTHLY :

- M1. Take a sample from the machine. Contact your distributor.

YEARLY or 2000 cycles (the shorter of the two terms)

- Y1. Have your Soluscope serviced as a preventive measure. Contact your distributor.
- Y2. Have your Soluscope qualified in accordance with European standards (15883-4) dealing with the washers. Contact your distributor.



If there are white deposits on the endoscope and in the tank, check the hardness of the water and contact your distributor.

Prolonged stop

- P.S.1: If your Soluscope has to be stopped for 72 hours or more, contact your distributor so that it gives you the instructions to follow to proceed to this prolonged stop.