



**SERIE 1 - SMALL IS BEAUTIFUL**

# Soluscope technology in a **compact device**

All-in-one automated  
endoscope reprocessor.

## MONITORING AT EACH LEVEL

The Serie 1 provides the right amount of single-use chemicals at each reprocessing cycle. It preserves the endoscope and offers a consistent process for user and patient safety.

The Soluscope Serie 1 integrates an automatic leak test to maintain the endoscope integrity during the cycle.

A secured sensor ensures that the lid is correctly closed when the device is running.

## SAFE PROCESS

The microbiological quality of the water is guaranteed by the 0.2µm terminal filter and an optional three-stage pre-filtration unit.

The Serie 1 features an automated self-disinfection cycle to avoid any risk of contamination.

The Serie 1 features an optional alcohol flushing and blowing phase to facilitate channel drying.

\*Soluscope Serie 1 PA Version



# A well thought out **design**

Simplicity and ergonomics.

## **SMART, EASY-TO-USE INTERFACES FOR FAST ADOPTION**

The Serie 1 touch screen provides an innovative and intuitive graphic interface. Just click on the screen to launch safe cycles and follow up in real time each step of the process.



## **TURNKEY ELECTRONIC TRACEABILITY AND DATA MANAGEMENT**

The Soluscope Serie 1 includes local data storage with optional barcode reader and printer.

It is compatible with our EP-Digital suite and enables electronic traceability, real-time monitoring of endoscope inventory and enables microbiological samplings and services records.



## **MINIMISED HANDLING TO REDUCE CROSS-CONTAMINATION RISKS**

Dedicated connectors reduce the need for endoscope handling and subsequently the risk of contamination.



## **EASY-ACCESS MAINTENANCE AND SERVICE**

The Serie 1 needs one power source and a water in/out connection to be installed.

Mobile and compact, it meets most space and accessibility requirements.

The chemical products and the terminal filter are accessible from the front panel of the device for quick and secured replacement.

# Validated efficacy

CE MARK / ISO 15883 -  
1 & 4 compliant.

## PROVEN SAFETY

Proven and tested according to EN ISO 15883-4:2018, Soluscope chemistry have been specifically designed for use on **Soluscope Serie 1** and is compatible with a large range of flexible endoscopes.

## CONNECTORS

Compatible with all Soluscope equipments, the connector is linked to the endoscope during all the reprocessing journey, from disinfection to storage.

This link ensures both connector and endoscope are at the same level of hygiene and provides an efficient and safe process before use on a patient.



### SOLUSCOPE PA

- Peracetic acid-based disinfectant
- Broad spectrum antimicrobial activity
- Compatible with a large range of flexible endoscopes



### SOLUSCOPE GTA

- Glutaraldehyde-based disinfectant
- Broad spectrum antimicrobial activity
- Compatible with a large range of flexible endoscopes



### SOLUSCOPE EZ


- Tri-enzymatic detergent
- Proven efficacy against biofilm and proteins

SERIE 1



Soluscope

“Ecolab provides  
a complete  
solution  
for flexible  
endoscope  
reprocessing”

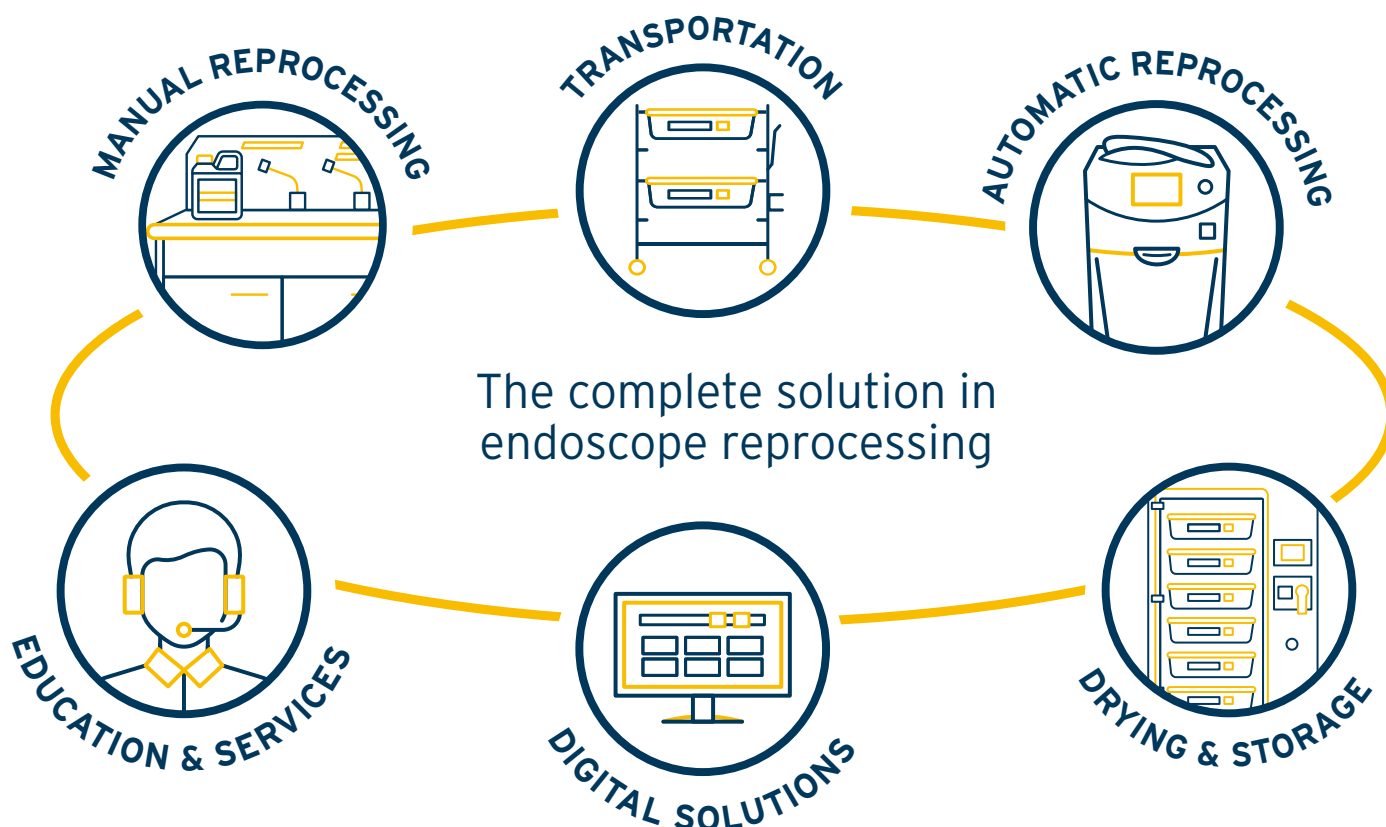

## THE ECOLAB SOLUTION

Ecolab provides a complete solution for flexible endoscope reprocessing.

It offers products for every step of the reprocessing cycle.

Each part is certified, easy to use and effective in order to reduce your daily load.



Non-contractual pictures - Soluscope reserves the right to make any changes to the specifications given in this document without notice or obligation.



**ECOLAB EUROPE**  
Richtstrasse 7  
8304 Wallisellen  
Switzerland



**SOLUSCOPE SAS**  
(An Ecolab Company)  
100, rue du Fauge  
Z.I Les Paluds - 13400  
Aubagne - France

**ECOLAB**<sup>®</sup>

[www.ecolab.com](http://www.ecolab.com)

**Soluscope**

[www.soluscope.com](http://www.soluscope.com)